

BELICHEM 180

BELICHEM 180 is a buffered liquid alkaline detergent for use in ultrasonic cleaners and automated washer machines for the removal of heavy contamination on surgical instruments, anesthetic utensils, and in laboratories.

Contains potassium silicate, potassium hydroxide & chelants and is universally suitable for the cleaning of surgical instruments. This buffered alkaline product is soft metal safe at recommended use dilutions. The high chelating properties allow for clean rinsing without the use of a rinse aid.

FEATURES

Contains chelating agents
Contains surfactants
Contains corrosion inhibitors
Highly concentrated
Low foaming
Biodegradable

BENEFITS

Excellent hard water tolerant
Optimal wetting and detergency
Safe on SS and most soft metals
Low cost per use
Effective with all automated washers
Non-polluting, ecology minded

DIRECTIONS FOR USE

Use 3.5 ml to 5 ml per liter of water or 1/3 ounce to 2/3 ounce per gallon of water. Recommended wash temperature is between 60°- 70°C. Dilutions may vary due to water conditions, soil load, and type of equipment. Contact your Belimed Sales Consultant for recommended usages.

PHYSICAL PROPERTIES

Appearance:	Clear, pale yellow liquid
Fragrance:	None
pH 1% @ 20°C:	11.9
SG @ 20°C	1.42
Foam:	Low
Available Alkalinity:	8.2

Available in 5, 15, 30, and 55 gallon containers