

BELICHEM 200

BELICHEM 200 is a mild liquid acid neutralizer agent/detergent for use in ultrasonic cleaners and automated washer machines for neutralizing alkaline residues left on surgical instruments, anesthetic utensils, laboratory equipment, and washer equipment.

Contains citric acid and a chelating agent. Product is compatible with stainless steel, anodized aluminum and most non-ferrous metals. This product will restore luster to tarnished equipment. Contains no surfactants.

FEATURES

Contains rinsing agent
Highly concentrated
Mild
Low foaming
Biodegradable

BENEFITS

Rinses spot free
Low cost per use
Effectively neutralizes instruments without harm
Effective with all automated washers
Non-polluting, ecology minded

DIRECTIONS FOR USE

Use 1 ml to 2 ml per liter of water or 1/8 ounce to 1/3 ounce per gallon of water. Dilutions may vary due to water conditions, soil load, and type of equipment. Contact your Belimed Sales Consultant for recommended usages.

For restoration cycle: Use 9 to 10 ml/L of water for 4 to 6 minutes at 50°C to 60°C. Immediately follow with a thorough series of post rinses.

PHYSICAL PROPERTIES

Appearance:	Clear, colorless liquid
Fragrance:	None
pH 1% @ 20°C:	2.5
SG @ 20°C	1.19
Foam:	None
Available Alkalinity:	None

Available in 5, 15, 30, and 55 gallon containers