

# BELILUBE 300 HC

BELILUBE 300 HC is a highly concentrated instrument lubricant for use in decontamination washing equipment and manual soak lubrication applications

Contains dual purpose corrosion inhibitors that protect all types of metal instruments, including carbon steel. The product is formulated not to interfere with steam sterilization and maintains free working instruments.

## FEATURES

Inhibits rust and corrosion  
Highly concentrated  
Non-silicone formulation  
Highly compatible with all types of water and instrument substrates  
Biodegradable  
Bacteriostatic

## BENEFITS

Extends instrument life  
Low cost per use  
Patient safe lubricant - will not cause abscesses  
Does not interfere with autoclaving or chemical sterilization  
Non-polluting, ecologically safe  
Shelf stable- inhibits bacterial growth

## DIRECTIONS FOR USE

Automated washers: Use .3 ml to .6 ml per liter or 1/24 ounce to 1/12 ounce per gallon of water. Dilutions may vary due to water conditions and type of equipment.

Manual: Add 1 part of lubricant to 18 parts of soft water. Mix thoroughly and immerse clean instruments, box locks open, into solution for 30 - 40 seconds and drain. Work tight instruments open and closed in solution until they operate freely.

Contact your Belimed Sales Consultant for recommended usages.

## PHYSICAL PROPERTIES

Appearance:	Opaque, white liquid
Fragrance:	None
pH @ 25°C:	6.5
SG @ 25°C:	0.982
Foam:	None
Available Alkalinity	None

Available in 2.5 gallon containers - 2 per case