

BELILUBE 300

BELILUBE 300 is a concentrated lubricant for use in decontamination washing machines and manual lubrication.

Contains dual purpose corrosion inhibitors that protect all types of metal instruments, including carbon steel. The product is formulated not to interfere with steam sterilization and maintains free working instruments.

FEATURES

Inhibits rust and corrosion
Highly concentrated
Non-silicone formulation
Highly compatible with all types
of water and instrument substrates
Biodegradable
Bacteriostatic

BENEFITS

Instruments with extended life
Low cost per use
Patient safe lubricant- will not cause abscesses
Does not interfere with autoclaving or
chemical sterilization
Non-polluting, ecology minded
Inhibits bacteria growth

DIRECTIONS FOR USE

Automatic Washer: Use .8 ml to 1.8 ml per liter or 1/8 ounce to 1/4 ounce per gallon of water.
Dilutions may vary due to water conditions and type of equipment.

Manual: Add 1 part of lubricant to 6 parts water. Mix thoroughly and immerse
clean instruments, box locks open, into solution for 30 to 40 seconds and drain.

Contact your Belimed Sales Consultant for recommended uses.

PHYSICAL PROPERTIES

Appearance:	Opaque, white liquid
Fragrance:	None
pH 1% @ 20°C:	6.1
SG @ 20°C	0.98
Foam:	None
Available Alkalinity:	None

Available in 5, 15, 30, and 55 gallon containers