

BELICHEM 900 HC

BELICHEM 900 HC is a highly concentrated neutral liquid detergent for use in automated wash equipment for the removal of moderate soil contamination on carts and accessories.

Contains a combination of surfactants, builders and chelating agents for good wetting ability and detergent penetration. Safe to use on stainless steel, anodized aluminum, zinc, rubber, latex and other delicate metals

FEATURES

Contains chelating agents
Contains surfactants
pH Neutral
Highly concentrated
Low foaming
Biodegradable

BENEFITS

Hard water tolerant
Optimal wetting action and detergency
Safe to use on all surfaces
Low cost per use
Effective with all automated washers
Non-polluting, ecologically safe

DIRECTIONS FOR USE

Automated cart washers: Use 1 ml to 1.7 ml per liter or 1/12 ounce to 1/6 ounce per gallon of water. Dilutions may vary depending on water quality, wash equipment, wash temperature and soil load.

Contact your Belimed Sales Consultant for recommended usages.

PHYSICAL PROPERTIES

Appearance:	Clear, blue liquid
Fragrance:	Fresh, floral
pH @ 25°C:	8.85
SG @ 25°C:	1.147
Foam:	Low
Available Alkalinity	Negligible

Available in 2.5 gallon containers - 2 per case